



HIRE EQUIPMENT SAFETY GUIDE

Shower Chair

PURPOSE

The Shower Chair provides a safe seated position whilst showering



FEATURES

- Plastic moulded seat which is comfortable and hygienic
- Arm rests assist with getting on and off the chair
- The height of the shower chair can be adjusted
- Non slip feet
- Strengthened frame for users needing extra room
- Seat height and angle adjusts to suit the individuals requirements
- One piece moulded supportive seat and backrest
- Small drainage slots allow water to escape and remove risk of entrapment
- SWL 200kg

INSTALLATION / SETUP

Adjust equipment to the appropriate height by depressing the silver button on the leg and sliding the leg pieces up and down until the button clicks into the appropriate hole. Repeat on all legs. Ensure all four legs are in contact with the floor to avoid rocking or unsteadiness

MAINTENANCE

The frame should be checked monthly for signs of rust, cracks, changes in the appearance of the rubber stoppers, or damage/bending in the metal. If any of these are present, stop use and stop use and contact your supplier

This product should be cleaned regularly with mild detergent unless otherwise stated by the manufacturer

SAFETY PRECAUTIONS

- Use two hands when transferring on and off the shower chair to avoid tipping
- This equipment is for sitting only. Do not stand on this equipment
- You should only use the equipment in the environment in which it is prescribed
- 125kg weight limit unless otherwise stated by the manufacturer
- Ensure the silver buttons on the height adjustable legs have slotted into the correct holes before sitting