



HIRE EQUIPMENT SAFETY GUIDE

Over Toilet Seat

PURPOSE

The Over Toilet Seat is designed to assist the user who has difficulty sitting down or standing up by raising the toilet seat height



FEATURES

- Aluminium construction for long life and no risk of rust
- Removable splash guard included
- Lift up seat that removes for cleaning
- Multi D leg adjustment ensures stability on uneven surfaces
- Push button height adjustment
- Non slip guides on base of legs
- SWL 125kg

INSTALLATION / SETUP

The over-toilet frame has height-adjustable legs to make your toilet seat higher making it easier to you to get on and off the toilet.

Set your desired height then position the frame directly over the toilet.

Always ensure that all 4 legs are stable on the floor before sitting.

When getting on and off the over toilet frame, always use two hands on the arm rests to stop you tipping of the chair.

MAINTENANCE

Use warm, soapy water to clean the chair or a cream cleansing agent for more stubborn stains. Rubber stoppers need to be checked and replaced regularly.

SAFETY PRECAUTIONS

- Putting weight through only one armrest may cause the over toilet frame to tip. Always push weight through both armrests
- Ensure all four legs are on an even surface
- Remove all mats, as they increase the risk of falling